


Trident RiX70DC - Error 06 Code

Error 06	High Voltage regulation error	<p>The system has detected that high voltage values are too high; the emission was interrupted.</p> <p>The image is not diagnostic.</p> <p>The system is blocked: switch off and on again.</p> <p>If the problem persists, call tech assistance to replace the driver board.</p>
-----------------	-------------------------------	---



Driver Board

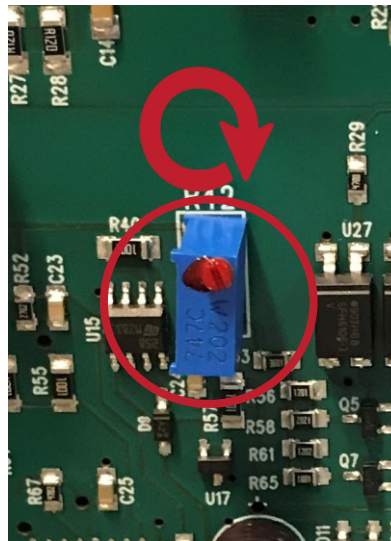

Trimmer R42

Adjustment Procedure

1. Be sure that the device has it plug connected directly to the electrical network, not through others secondary equipment or surge protectors.
2. Be sure that the electrical net is 120V with no voltage fluctuations.
3. Let's try to reduce the tension by acting on the trimmer R42. Have it turned clockwise, one complete circle (360°) means about 1 kV. Test if the error code was eliminated after each 1 kV decreased. If necessary proceed with this adjustment "1 kV at the time" but with a maximum of 3 kV (three complete turns/circles).
4. If after this procedure the issue persists, contact with our support service.

WARNING: THIS PROCEDURE MUST BE DONE FOR A SPECIALIZED TECHNICIAN. WE ARE NOT RESPONSIBLE FOR ANY ACIDENT OR DAMAGE DURING THE PROCEDURE