



XRD STE23 LCD Class B Sterilizer Work Chart

Form 1 Work mode chart

	B cycle	18min Bcycle	Unwrappd cycle	RUB & PLAS cycle	B&Dtest	Vacuum test	Clean test
Sterilizing temperature	134	134	134	121	134	---	---
Sterilizing pressure	2.10	2.10	2.10	1.10	2.10	---	---
Class	B	B	S	B	B	---	---
Pre-heating	0-7min	0-7min	0-7min	0-7min	0-7min	---	---
Vacuuning time	10min	10min	4min	10min	10min	4min	Clean 20 min
Admission time	5min	5min	5min	5min	5min	---	---
Sterilizing time	4min	18min	4min	16min	3.5min	Keep pressure 10min	---
Drying time	10min	10min	10min	10min	1min	---	Drying 10 min
Quick drying time	5min	5min	5min	5min	---	---	---
Total time	36(31)min	50(45)min	30(25)min	48(43)min	19.5min	15min	30min

Note: The figures in the chart are only for reference, the sterilization effect verified by user, and the user can choose the sterilization time according to the verification results. Which cycle should be adopted is according to the type of the instruments.

Working chart

